

# THE ISOLITE GUIDE TO STRESS-FREE EXTRACTIONS

Follow these steps to success

## CHOOSE THE RIGHT SIZE MOUTHPIECE

If you can place 3 fingers into the inter-incisal opening of your patient's mouth, use a Medium mouthpiece. If it's only 2 fingers, choose a Small mouthpiece. If you're in between, go with the smaller mouthpiece.



## EXPLAIN THE BENEFITS TO YOUR PATIENT

Other dentists have shared that they tell their patients the Isolite is going to help them keep their mouth open so you can finish sooner than if you didn't use the mouthpiece.

## REMIND YOUR PATIENTS THAT YOU CARE ABOUT THEM

Tell your patients you're using the Isolite because you care about them and don't want them to taste any blood, and that you're doing everything you can to protect their airway.



## PRACTICE MOUTHPIECE PLACEMENT

We know that mouthpiece placement takes getting used to. Practice on yourself and your team to get comfortable so your next extraction is stress-free and smooth sailing.