INSTRUCTIONS FOR USE

ISOVAC®

Dental Isolation Adapter

EN

CS

DA

DE

EL

ES

FI

FR

IT

JA

NL

NO

PL

PT

SL

SV

TABLE OF CONTENTS
OBSAH
INDHOLDSFORTEGNELSE
INHALTSVERZEICHNIS
ΠΙΝΑΚΑΣ ΠΕΡΙΕΧΟΜΕΝΩΝ
CONTENIDO
SISÄLLYSLUETTELO
TABLE DES MATIÈRES
INDICE
目次
INHOUDSOPGAVE
INNHOLDSFORTEGNELSE
SPIS TREŚCI
ÍNDICE
KAZALO VSEBINE
INNEHÅLLSFÖRTECKNING

ISOVAC®

Dental Isolation Adapter

Isovac® Dental Isolation Adapter: Instructions for Use

EN

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[REF] EIV0107

[REF] EIV0108

This system is intended for use with Mouthpieces developed by Zyris, Inc. These Mouthpieces are available in a wide range of sizes for effective isolation.

Each Isovac Dental Isolation Adapter is comprised of one main component: Control Head

Control Head

[REF] PIV0101

The Control Head includes two Vacuum Levers, a Right (P/N: 20-0405) and a Left (P/N: 20-0406), to adjust suction in the upper and lower areas of the Mouthpiece.

Suction Line Plug

[REF] PIL0206

O-ring Maintenance Kit

[REF] AIL0226

Double End Tube Brush

[REF] AIL0225

Lever Set

[REF] PIV0102

Sold separately

Mouthpiece Lubricant

[REF] AIL0202

Sold separately

Hose Connectors

[REF] AIL0227

[REF] AIL0231

[REF] AIL0232

[REF] AIL0233

[REF] AIL0234

[REF] AIL0235

[REF] AIL0236

Sold separately

Some chairs may require a hose connector to install the Isovac onto the High Volume Evacuation (HVE) valve or Saliva Ejector hose.

Mouthpieces

(Sold separately)

-  CIL0501 Pediatric
-  CIL1101 Extra Small
-  CIL0601 Small
-  CIL0701 Medium
-  CIL0801 Medium DV
-  CIL0901 Large

Mouthpieces are single use only and available in various sizes. The Mouthpieces are constructed of a soft, flexible polymer and are not made with natural rubber latex. They do not contain phthalates or BPAs.

Intended Use

Isovac is a dental isolation adapter for clinical use by dental professionals. The adapter is designed to isolate two quadrants at once, simultaneously providing tongue and cheek retraction and continuous suction. The retraction and continuous suction are the important functions that control the oral environment, reducing contamination and improving bond strength while protecting the airway and soft tissue.

Contraindications

No known contraindications. Do not use

without reviewing how to select the correct Mouthpiece size for each patient. Do not use if unable to position the Mouthpiece correctly.

Side Effects

Use of the Mouthpiece may trigger a pharyngeal reflex (laryngeal spasm) in hypersensitive patients, or if it is not placed correctly within the oral cavity.

How Supplied

Isovac

The Isovac Dental Isolation Adapter contains: Control Head, Suction Line Plug, Double End Tube Brush, O-ring Maintenance Kit and Instructions for Use.

Mouthpieces

Mouthpieces (sold separately) are supplied in convenient operatory boxes. Each Mouthpiece is individually packaged.

Installation

Attaching without a Connector

Isovac is designed to install easily onto an existing High Volume Evacuation (HVE) Valve or Saliva Ejector in most dental operatories. To install, detach the HVE Valve or Saliva Ejector from the tailpiece of the evacuation hose. Next, attach the Isovac directly onto the tailpiece. Press firmly in place (Figure 1).

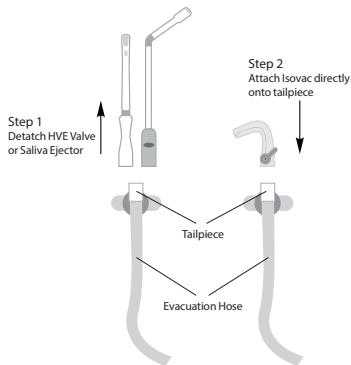
Figure 1. Installation without a Connector

Attaching with a Connector

For some dental units, a connector (sold separately) may be required to attach the Isovac to the tailpiece of the evacuation hose. To install, detach the HVE Valve or Saliva Ejector from the tailpiece of the evacuation hose. Next, attach the connector onto the tailpiece. Then, attach the Isovac onto the connector. Press firmly in place (Figure 2).

To find the correct connector, identify the connection type for your dental unit (Table 1). Contact your Customer Service Representative if your connection type is not shown.

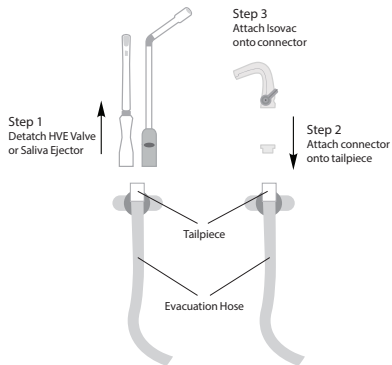
Figure 2. Installation with a Connector

Connection Type:	A-dec	Pelton & Crane Style	Sirona	Heka	XO Care	Anthos, Castellini, Finndent, Kavo, OMS, Planmeca, Stern Weber
Connector:						
Part Number:	REF AIL0227	REF AIL0231	REF AIL0233	REF AIL0234	REF AIL0235	REF AIL0236
Attaches to:	HVE	HVE	Saliva Ejector	Saliva Ejector	Saliva Ejector	Saliva Ejector

Table 1. Connection Types and Connectors

Attaching Directly onto HVE Valve / Saliva Ejector Control Heads

To attach the Isovac directly onto HVE Valve / Saliva Ejector Control Heads, use the 11 mm Connector (sold separately) (Table 2). To install, detach the vacuum tip from the Control Head. Next, attach the 11 mm Connector onto the Control Head. Then, attach the Isovac onto the 11 mm Connector. Press firmly in place (Figure 3).

Connector:	Connection Type:	Part Number:	Attaches to:
	11 mm Connector	 AIL0232	HVE Valve / Saliva Ejector Control Head

Table 2. 11 mm Connector.

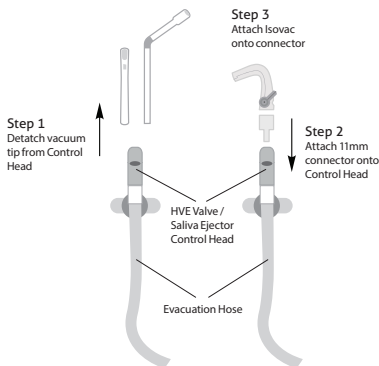
Figure 3. Installation Directly onto HVE Valve / Saliva Ejector Control Heads

How to Operate

Using the Dual Vacuum Levers

Dual Vacuum Levers adjust suction intensity in both the upper and lower quadrants. To adjust, rotate the left and right Vacuum Levers to the desired suction intensity for each quadrant. To shut off suction, rotate the levers to the most downward position.

Selecting the Mouthpiece

To select the correct Mouthpiece size, measure the patient's interincisal opening. If between sizes, you may select the smaller size.

Mouthpiece Size	Approx. Interincisal Opening
Pediatric (PED)	< 30 mm
Extra Small (XSM)	< 30 mm
Small (SML)	30 mm
Medium (MED)	30-45 mm
Medium Deep Vestibule (MDV)	30-45 mm
Large (LRG)	> 45 mm

Table 3. Mouthpiece Sizes

The Finger Method

The finger method may also be used to select the appropriate Mouthpiece size (Figure 5).

Attaching the Mouthpiece

Spray water into the Mouthpiece Sleeve (Figure 6). Insert Control Head into the Mouthpiece Sleeve until Locking Barbs engage (Figure 7).

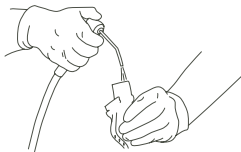
Figure 6. Spraying Mouthpiece Sleeve

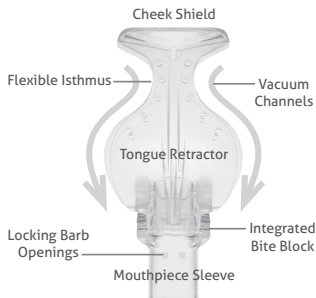
Figure 4. Mouthpiece

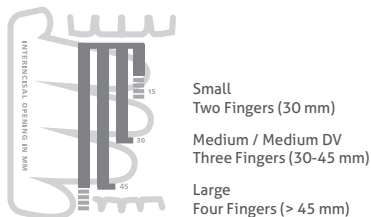
Figure 5. The Finger Method

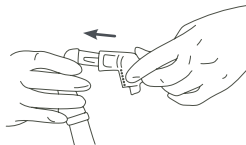
Figure 7. Attaching Mouthpiece

Placing the Mouthpiece

Prior to placing the Mouthpiece within the patient's oral cavity, moisten the Mouthpiece and the patient's lips.

Step 1

Hold the sides of the Control Head with your index finger and thumb. With your other hand, fold the Cheek Shield onto the Tongue Retractor. Gently slide the folded mouthpiece into the buccal vestibule on the side to receive treatment. Angle the mouthpiece to allow the lower edge of the Tongue Retractor to move along the buccal edge of the teeth.

Step 2

Move the bite block onto the occlusal surface of the teeth, just distal to the mandibular cuspid. Instruct the patient to "rest gently" on the bite block to secure.

Step 3

Place the Cheek Shield into the buccal vestibule. Move the Isthmus of the Cheek Shield behind the maxillary tuberosity. Adjust the Tongue Retractor in the lingual vestibule as needed. Move the bite block distally to provide more vertical working room.

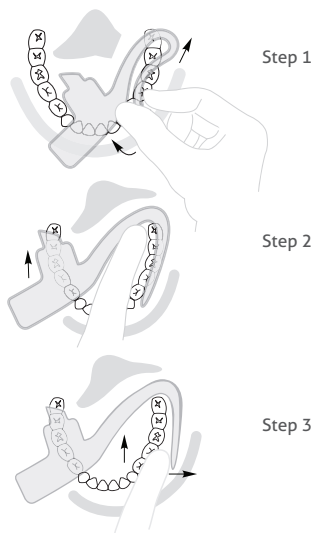
Figure 8, 9, & 10. Mouthpiece Placement

Disassembly of the Control Head

Step 1

Holding the head of the vacuum hose, pull to remove the Control Head. Do not bend. When Isovac is removed, the Suction Line Plug (PN: PIL0206) can be used to plug the suction hose.

Step 2

Remove the Vacuum Levers by rotating the levers to "off" position and keep pushing gently until each lever pops out, or gently pry them out (Figure 11).

Step 3

Separate the upper and lower halves of the Control Head by pulling the two halves apart from the hinges.

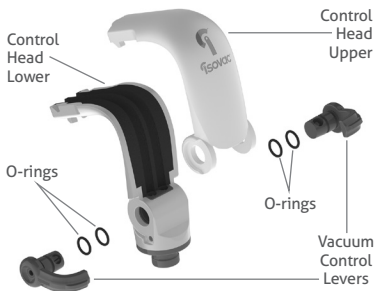
Figure 11. Isovac Dental Isolation Adapter Components

Cleaning and Sterilization

Cleaning and Sterilization of the Control Head

All components of the Isovac can be sterilized in an autoclave. Zyris recommends sterilization between each patient.

Remove biomatter and other debris from the suction channels using the provided Double End Tube Brush with a neutral pH (6.5-7.5) enzymatic cleaning solution.

After cleaning, rinse with tap water, dry, and place components in an autoclave pouch to be sterilized in a steam autoclave. Use validated sterilization cycles (Table 4).

Validated Sterilization Cycles

Cycle	Temperature	Min. Exposure Time	Min. Drying Time
Standard	132 °C / 270 °F	15 min	15 min
Prevacuum	132 °C / 270 °F	4 min	20 min
Prevacuum	134 °C / 273 °F	3 min	20 min

Table 4. Validated Sterilization Cycles

Maintenance

Control Head

After sterilization, lubricate the Vacuum Lever O-rings with the provided Dow 111 O-ring lubricant. Check the inside of the Lever for L (Left) and R (Right) for correct placement (Figure 12).

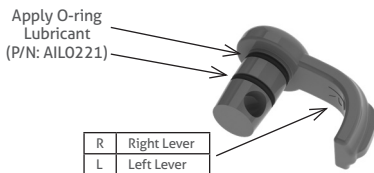
Figure 12. Vacuum Lever to Lubricate

Figure 13. Retainer Ring to Lubricate

Reporting Issues

The user and/or patient should report any suspected serious incident related to the

device by informing the manufacturer and the competent authority of the member state in which the serious incident occurred.

Warnings

⚠ **WARNING:** Mouthpieces are single use only. Reuse can result in the transmission of infectious pathogens. Mouthpieces must be discarded after use. Mouthpieces cannot be safely sterilized after use.

⚠ **WARNING:** Do not use if the individual Mouthpiece package has been previously damaged or opened. Non-sterile Mouthpieces are individually packaged in polyethylene bags.

⚠ **WARNING:** Be sure to use established protocols and patient monitoring when using the Mouthpiece in conjunction with sedation dentistry. Patient management by a certified anesthesiologist is strongly recommended.

Precautions

⚠ **CAUTION:** When performing dental procedures requiring high levels of blood evacuation, it is recommended that the Mouthpiece be intermittently flushed with water to prevent coagulation in the Mouthpiece to avoid loss of suction.

⚠ **CAUTION:** Do not autoclave the Control Head at temperatures above 135 °C; higher temperatures could

damage components and void your product warranty.

⚠ **CAUTION:** Failure to maintain properly lubricated O-rings can result in damage to the Isovac that is not covered by product warranty.

⚠ **CAUTION:** With regular use, cleaning, and maintenance, the device is designed to last for years. Check the device for wear prior to use to make sure that it is adequate for use.

⚠ **CAUTION:** Dispose of the system components in accordance with federal, state, and local laws.

Limited 1-Year Warranty

Zyris warrants this product to be free from defects in material and workmanship for a period of one year from the date of original purchase. If the product exhibits such a defect, Zyris will, at its option, repair or replace it without costs for parts and labor. The product must be returned by the customer to Zyris in accordance with current Return Material Authorization procedures. Proof of date of original purchase may be required. This warranty does not cover finishes or normal wear, nor does it cover damage resulting from accident, misuse, dirt, tampering, unreasonable use, failure to provide reasonable and necessary maintenance, service performed or attempted by unauthorized individuals or service agencies, or devices that have

been otherwise modified. All implied warranties, including any implied warranty or merchantability or fitness for any particular purpose, are limited in duration to one year from the date of receipt of the product. In no event will Zyris be responsible for consequential damages resulting from the use of this product.

Instructions for Use may also be found at: https://zyris.intl_ifu

Guide to Symbols

Catalog number

Non-sterile

Caution

Date of
manufacture

Do not
re-use

Do not use
if package is
damaged

Consult
Instructions
for Use

Manufacturer

15° 30°
Storage
Conditions

Medical
Device

European
Authorized
Representative

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All rights reserved.
© 2025 Zyris, Inc.